

Symbol Legend



CONSULT INSTRUCTIONS
FOR USE



CAUTION



MANUFACTURER



TEMPERATURE LIMITS



European Pharma Group BV



ISO 13485 certified

Euro Offices Building 3rd Floor
Beechavenue 127, 1119 RB
Schiphol-Rijk, The Netherlands

www.insujet.com | info@insujet.com

CE 0459

017403-EN-1.0
27 Sept 2016

insuJet™


Insulin - Jet Administration System

Comfort Ring information

- Please read the complete instructions for use booklet inside the InsuJet™ package before using the InsuJet™ system.
- Keep the original InsuJet™ instructions for use in safe custody.




Using a Comfort Ring

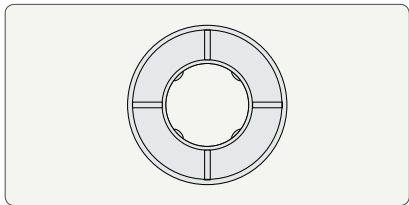

The Comfort Ring is a part that can optionally be used to reduce the pressure of the Nozzle on relatively soft or sensitive administration areas. Place the Comfort Ring on top of the Nozzle, just before administration.

1. Make sure that the rounded side of the Comfort Ring is facing outward.

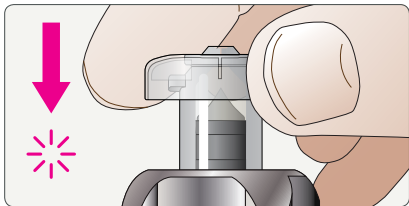

2. Hold the InsuJet™ by its Front and push the Comfort Ring on top of the Nozzle until it snaps in place.

NOTES:

- A Comfort Ring can be used multiple times. It is recommended to replace the part once the fitting with the Nozzle becomes loose.
- For hygienic reasons, be careful not to touch the Nozzle tip with your fingers when installing the Comfort Ring.