

Symbol Legend



CONSULT INSTRUCTIONS
FOR USE



CAUTION



MANUFACTURER



TEMPERATURE LIMITS



European Pharma Group BV

Euro Offices Building 3rd Floor
Beechavenue 127, 1119 RB
Schiphol-Rijk, The Netherlands

www.insujet.com | info@insujet.com



017403-EN-2.1

14 Apr 2018

insuJet™


Insulin - Jet Administration System

Comfort Ring information

- Please read the complete instructions for use booklet inside the InsuJet™ package before using the InsuJet™ system.
- Keep the original InsuJet™ instructions for use in safe custody.




Using a Comfort Ring

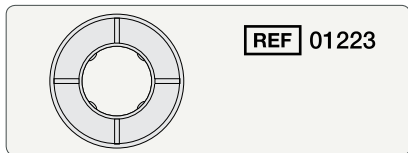

The Comfort Ring is an InsuJet™ accessory. It is recommended to use a Comfort Ring, in case you feel discomfort when pushing the Nozzle against the skin.

To install a Comfort Ring, hold the device by its Front and place the Comfort Ring on top of the Nozzle. Make sure that the round side of the Comfort Ring is facing outward.

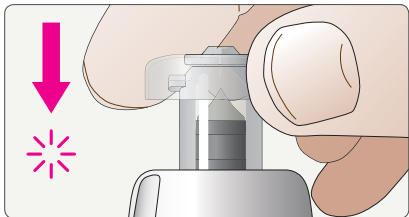

Push the Comfort Ring on top of the Nozzle until it snaps in place.

NOTES:

- A Comfort Ring can be used multiple times. It is recommended to replace the part once the fitting with the Nozzle becomes loose.
- For hygienic reasons, be careful not to touch the Nozzle tip when installing the Comfort Ring.
- Temperature limits for use:

